# ReGENERERA®

## therapeutic lens

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**ENGLISH** 1 soft therapeutic contact lens for eye bandage (ionic material G-72 HW H<sub>2</sub>O 72%) in saline solution buffered with Aloe Alvera®. **DIRECTIONS FOR USE** This leaflet contains information for users of therapeutic contact lenses REGENERERA therapeutic lens® on how to wear and take care of their lenses. **Description and wearing** The therapeutic contact lens REGENERERA therapeutic lens® can be used only under strict medical supervision from the ophthalmologist or the optician. REGENERERA therapeutic lens® does not have a refractive function. Two models have been developed: REGENERERA corneal therapeutic lens® to facilitate corneal re-epithelialisation after refractive surgery or in the case of scratching or injury of the corneal surface; REGENERERA scleral therapeutic lens® as an ocular bandage after surgery or in the case of pathologies that require limbal coverage for wider protection of the eye surface. The ionic nature of the material allows better interaction with the tear film and optimum passage of eye drops through the lens.<sup>(1)</sup> REGENERERA therapeutic lens® can be worn up to a maximum of 30 consecutive days. Do not utilize after 30 days. The lens is soddien in aloe solution Alvera® that has soothing and anti-inflammatory action, and it also helps the re-epithelialization process.<sup>(1,2)</sup> **How to wear and remove the contact lens: WEARING** Always wash your hands thoroughly with a mild soap and rinse completely. Place the lens in the palm of your hand. Make sure the lens is in the correct position.<sup>(1)</sup> Inspect the lens before inserting it and do not insert if it is scratched or damaged. Please check to see that the package is intact, not open or damaged, because this is a condition of warranty, sterility and conformity of the product. If package is damaged, please return it to the reseller. After opening the container, check to ensure the lens inside floats in the solution. Take the lens with the fingertips or with a sterile pinper. You are recommended to handle it carefully and not to damage it. Lower your lower eyelid and lift your upper eyelid and carefully position the lens on the iris. Gently let go of your eyelid. Now you have to check: • the correct position of the lens. If you need, you can place it in the correct position massaging your eyelids, or, with the open eye, move the not-centered lens on the cornea with a light pressure on the upper or lower eyelid. • be sure that the lens is correctly oriented. On the contrary, remove the lens and rinse with the specially provided solutions, check the orientation through the correct conformity and wear it another time. • be sure to have followed all hygienic and sanitary regulations regarding the washing of the hand and the correct conservation of the lens. If you do not have followed these directions, please use a new lens. In case of non-movement of the lens in the eye or if it is uncomfortable, please contact eye care practitioner for a lubricating and humectant solution. IF THIS SITUATION DOES NOT IMPROVE, PLEASE CONTACT YOUR SPECIALIST. **REMOVING** To remove the lens, make sure your hands are clean and dry, then use a finger to slide the lens downwards onto the white of the eye and remove it. Use eye drops if the lens is difficult to remove. **FURTHER PRESCRIPTIONS AND PRECAUTIONS** • Please do not use the lens after its use-by date • Please use only eyewashes that are compatible with the use of the contact lens, some ocular drugs can compromise the tolerability of the lenses • Some drugs (antihistamine, decongestant, diuretic, muscle relaxant, with a tranquilizing effect or for the syndromes of the movement) can cause dryness in the eye, increase the sensitivity and blur the vision • Some drugs can modify the sensitivity and change the quality of the vision • Please do not use spray or aerosol if you are wearing the lenses • The REGENERERA therapeutic lens® is not recommended for people who are sensitive or intolerant to distilled water of aloe, because this additive is present in the preservation's solution of the product.

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